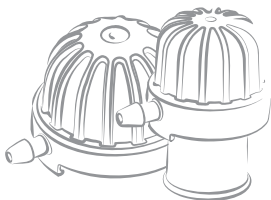
FREEVENT[®]

XtraCare[™] and XtraCare[™] *Mini*
O₂ Adaptor and O₂ Adaptor *Mini*

Freevent XtraCare
O₂ Adaptor

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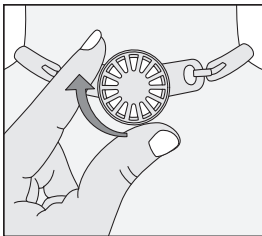
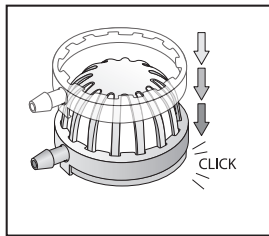
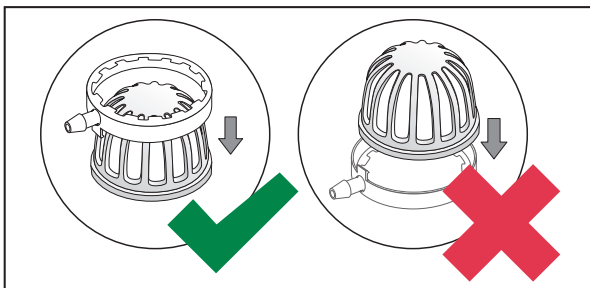
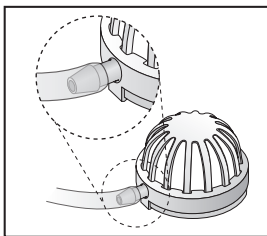
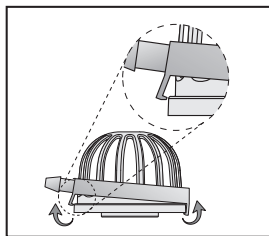
5.

Prescription information

CAUTION: United States Federal law restricts this device to sale, distribution and use by or on order of a physician or a licensed practitioner. The availability of this product without prescription outside the United States may vary from country to country.

Disclaimer

Atos Medical offers no warranty – neither expressed nor implied – to the purchaser hereunder as to the lifetime of the product delivered, which may vary with individual use and biological conditions. Furthermore, Atos Medical offers no warranty of merchantability or fitness of the product for any particular purpose.

Contents

EN - ENGLISH.....	5
DE - DEUTSCH	9
NL - NEDERLANDS	14
FR - FRANÇAIS	19
IT - ITALIANO	24
ES - ESPAÑOL	29
PT - PORTUGUÊS	34
SV - SVENSKA.....	39
DA - DANSK	44
NO - NORSK	49
FI - SUOMI	54
IS - ÍSLENSKA.....	59
ET - EESTI	64
LT - LIETUVIŲ KALBA	69
CS - ČESKY	74
HU - MAGYAR.....	79
SK - SLOVENČINA	84
SL - SLOVENŠČINA	89
PL - POLSKI.....	94
RO - ROMÂNĂ	99
HR - HRVATSKI.....	104
SR - SRPSKI.....	109
EL - ΕΛΛΗΝΙΚΑ	114
TR - TÜRKÇE.....	119
HY - ՀԱՅԹԻՐԵՆ	124
RU - РУССКИЙ	130
MS - BAHASA MELAYU	135
JA - 日本語	140
KO - 한국어.....	145
ZHTW - 繁體中文	150
HE - עברית.....	159
AR - العربية.....	164

Freevent® XtraCare™ System

1. Description of the device

Freevent® XtraCare™ System consists of Freevent® XtraCare™/Freevent® XtraCare™ Mini and Freevent® O₂ Adaptor/Freevent® O₂ Adaptor Mini.

2. Disposal

Always follow medical practice and national requirements regarding biohazard when disposing of a used medical device.

3. Reporting

Please note that any serious incident that has occurred in relation to the devices shall be reported to the manufacturer and the national authority of the country in which the user and/or patient resides.

Freevent XtraCare and Freevent XtraCare Mini

1. Intended use

Freevent XtraCare/Freevent XtraCare Mini are single use Heat and Moisture Exchangers with electrostatic filter (HMEF) that condition and filter inhaled air in patients spontaneously breathing through a tracheostoma.

2. CONTRAINDICATIONS

Freevent XtraCare/Freevent XtraCare Mini are contraindicated for patients who:

- are under any form of mechanical ventilation.
- are unable to handle or remove Freevent XtraCare/Freevent XtraCare Mini themselves when needed, and who are not under constant supervision of a clinician or a trained caregiver.
- have a low tidal volume, as the extra dead space may cause CO₂ (Carbon dioxide) retention.

Freevent XtraCare/Freevent XtraCare Mini are contraindicated if single lumen tracheal tube cannot be reinserted by patient or caregiver after accidentally removed.

3. Description of the device

The device is available in two versions:

- Freevent XtraCare.
- Freevent XtraCare Mini

Freevent XtraCare/Freevent XtraCare Mini is a Heat and Moisture Exchanger combined with an electrostatic filter (HMEF). The HME is impregnated with a

hygroscopic salt and conditions the inhaled air. The electrostatic filter provides effective filtration and reduces the inhalation and exhalation of small airborne particles such as viruses, bacteria, pollen and other particles through the tracheostoma, regardless of the direction of airflow through the device. NOTE: Since there are other pathways for pathogens to enter the human body, Freevent XtraCare/Freevent XtraCare Mini can never guarantee total protection. Freevent XtraCare/Freevent XtraCare Mini has a 15 mm ISO connector for connection to a tracheostomy tube. For supplemental oxygen the compatible O₂ adaptor can be connected to Freevent XtraCare/Freevent XtraCare Mini. See section 3 of Freevent O₂ Adaptor/Freevent O₂ Adaptor Mini.

4. Technical data

	Freevent XtraCare	Freevent XtraCare Mini
Dead space	7.4 ml	6.1 ml
Tidal volume range	>50 ml	30-250 ml
Pressure Drop* at* - 10 l/min - 30 l/min - 60 l/min	- 70 Pa 150 Pa	45 Pa 150 Pa -
Moisture loss* at - VT= 250 ml - VT= 500 ml - VT= 1000 ml	- 18 mg/l 19.5 mg/l	13.5 mg/l - 21.5 mg/l
Moisture loss* with O ₂ adaptor at following flow rates: - VT= 250 ml, O ₂ = 4 l/min - VT= 500 ml, O ₂ = 15 l/min - VT= 1000 ml, O ₂ = 15 l/min	- - 23.8 mg/l	14.7 mg/l 16.3 mg/l -
Filtration efficiency** - Virus (VFE) - Bacteria (BFE)	At 30 l/min >99% >99%	At 15 l/min >98% >99%

* According to ISO 9360

** Test method adapted from ASTM F2101

5. WARNINGS

Do not reuse Freevent XtraCare/Freevent XtraCare Mini on another patient. Re-use in another patient may cause cross-contamination which may lead to serious infection.

6. PRECAUTIONS

- Do not use a damaged or disassembled Freevent XtraCare/Freevent XtraCare Mini as this may cause aspiration of small parts.
- Do not use a dirty or contaminated Freevent XtraCare/Freevent XtraCare Mini as this may cause an infection.
- Do not wash and re-use Freevent XtraCare/Freevent XtraCare Mini. Washing the devices impairs the filtering- and HME functions.
- Do not use Freevent XtraCare/Freevent XtraCare Mini longer than 24 hours. The risk of potential infections may increase with the time of use due to pathogens (such as viruses, bacteria or other microorganism) of the foam.
- Do not administer medicated nebulizer treatment over Freevent XtraCare/Freevent XtraCare Mini since the medication can become deposited in Freevent XtraCare/Freevent XtraCare Mini.
- Do not use humidifiers or heated humidified oxygen over Freevent XtraCare/Freevent XtraCare Mini since the device will become too wet and may result in an increased work of breathing.
- A high secretion load may indicate more frequent replacement of Freevent XtraCare/Freevent XtraCare Mini to avoid higher breathing resistance and blocking.

7. Instructions for use

Freevent XtraCare/Freevent XtraCare Mini is for single use and must be replaced at least every 24 hours, or more often as needed (e.g. if clogged by mucus or wet).

To use Freevent XtraCare/Freevent XtraCare Mini, gently attach Freevent XtraCare/Freevent XtraCare Mini to the tube using a twisting motion (Fig. 1). To remove Freevent XtraCare/Freevent XtraCare Mini from the tracheostomy tube, hold the tracheostomy tube in place and gently pull Freevent XtraCare/Freevent XtraCare Mini using a twisting motion.

8. Compatible products

Freevent O₂ Adaptor / Freevent O₂ Adaptor Mini:

Freevent O₂ Adaptor / Freevent O₂ Adaptor Mini are accessories for patients requiring supplemental oxygen. The adaptor connects oxygen tubing of 1/8 inch (3.2 mm) diameter to the oxygen port.

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1. Intended use

Freevent O₂ Adaptor and Freevent O₂ Adaptor Mini are single use accessories used together with Freevent XtraCare or Freevent XtraCare Mini respectively.

The devices are intended to enable additional oxygen supply for patients breathing through a tracheostoma during use of Freevent XtraCare and Freevent XtraCare Mini.

2. Description of the device

Freevent O₂ Adaptors are accessories that fit Freevent XtraCare/Freevent XtraCare Mini. They are clicked over the base of Freevent XtraCare/Freevent XtraCare Mini and the combined Freevent XtraCare/Freevent XtraCare Mini is then attached to the patient's tracheostomy tube, or compatible products. Additional oxygen can then be supplied via the oxygen port of the O₂ Adaptor. Freevent O₂ Adaptors are single use devices and should be replaced if they become dirty, or at least every 24 hours.

3. Instructions for use

Attach and detach the Freevent O₂ Adaptor/Freevent O₂ Adaptor Mini to Freevent XtraCare/Freevent XtraCare Mini as shown in (Fig. 2). The Freevent O₂ Adaptor/Freevent O₂ Adaptor Mini should be placed over the top of the Freevent XtraCare/Freevent XtraCare Mini and clicked into place. The Freevent O₂ Adaptor/Freevent O₂ Adaptor Mini should not be placed from the bottom side of Freevent XtraCare/Freevent XtraCare Mini as shown in (Fig. 3).

Attach the oxygen tubing to the Freevent O₂ Adaptor/Freevent O₂ Adaptor Mini as shown in (Fig. 4). Make sure the oxygen tube goes beyond the conical edge of the connector (Fig. 5). Flow rates of up to 15 l/min can be used. However, the HME effect will be reduced as the oxygen flow increases. At a typical flow of 3-4 l/min, the HME performance is minimally affected but at 15 l/min HME cannot completely compensate for the extra drying caused by supplementary oxygen.

Freevent O₂ Adapter/Freevent O₂ Adaptor Mini are for single use and must be replaced at least every 24 hours, or more often as needed.

4. PRECAUTIONS

- Please ensure oxygen flow rate has been prescribed by a qualified clinician before administering, see section 4. Technical data of Freevent XtraCare and Freevent XtraCare Mini.
- The adaptor is for single use and should be replaced at least every 24 hours, or when it becomes dirty, contaminated or shows any signs of damage.
- Freevent O₂ Adaptor Mini is small and may present a hazard for small children if ingested.

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© Coloplast A/S. All rights reserved.
Atos and the Atos Medical logo are trademarks of Coloplast A/S.
Atos Medical AB